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The screenshot displays the Ventana VANTAGE Digital Pathology interface. The top navigation bar includes the Ventana logo and the text "Microtomy". On the right side of the navigation bar, there is a "Welcome Admin" message, a "Logout" button, and a help icon (?).

The main content area is divided into several sections:

- Accession Number:** BM09-146 3A  
Block: 3 of 3 | Slide(s): 1
- Case ID:** BM09-146
- Patient Name:** Allen, Lisa
- Date of Birth:** 1/1/1943
- Gender:** F
- Tissue Type:** Bone Marrow, Aspirate
- Special Instructions:** Embed on Side
- Quality Issues:** Leaking container


A thumbnail image of a slide is displayed in the upper right quadrant, showing a QR code, the text "H&E 1/1", and the print date "Printed: 2009/10/06 6:15 AM".

At the bottom of the interface, there are navigation buttons for "Microtomy", "Work List", and "History". On the right side, there are icons for a grid view, a printer, and the current date and time: "Oct 06, 2009 6:15 AM".

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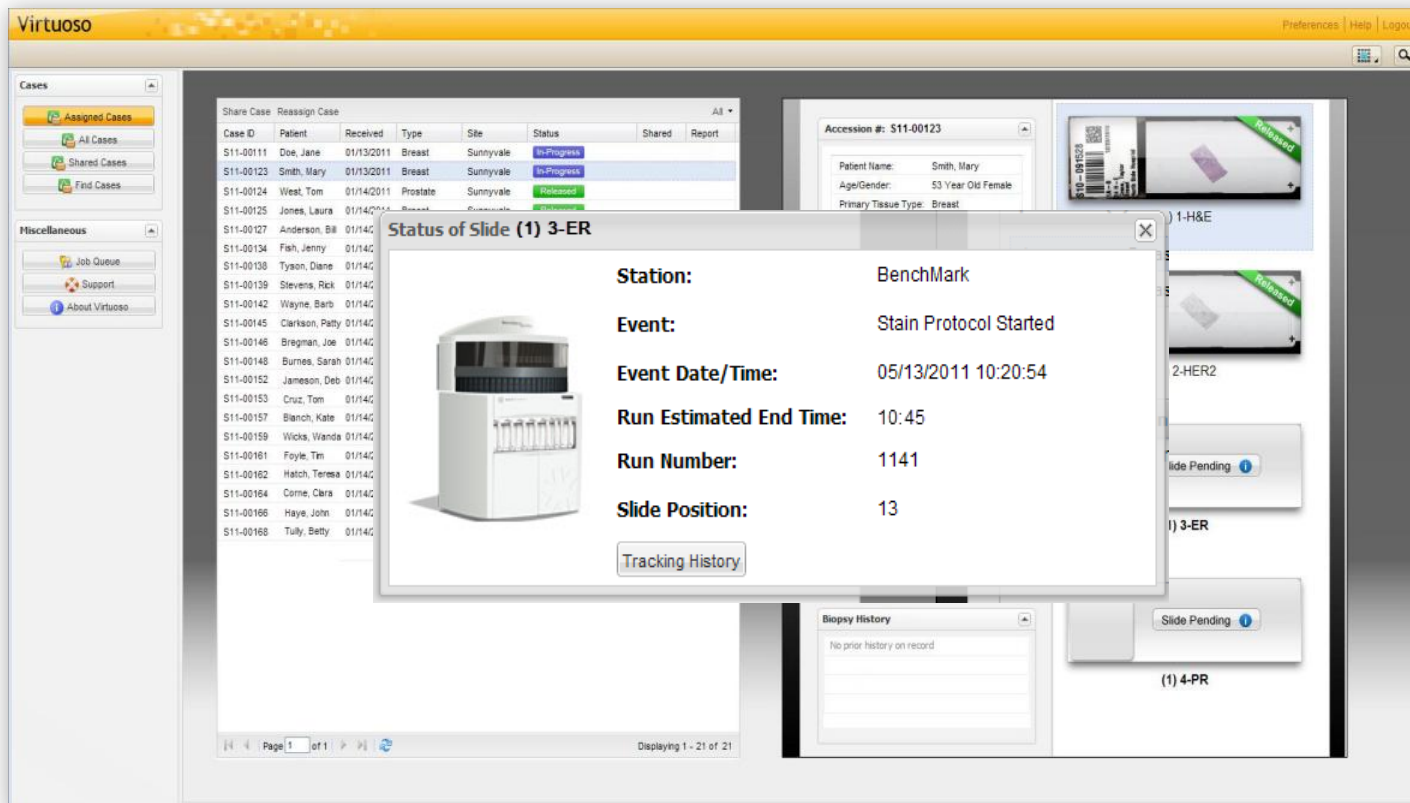The screenshot displays the Virtuoso Digital Pathology interface. On the left, there are navigation panels for 'Cases' (Assigned, All, Shared, Find) and 'Miscellaneous' (Job Queue, Support, About). The main area shows a table of cases with columns for Case ID, Patient, Received, Type, Site, Status, Shared, and Report. A modal window titled 'Status of Slide (1) 3-ER' is open, showing details for a slide at the BenchMark station, including the event 'Stain Protocol Started' on 05/13/2011 at 10:20:54. A 'Tracking History' button is visible in the modal. Below the modal, a 'Biopsy History' section shows 'No prior history on record'. On the right, there are images of slides with 'Released' labels and a 'Slide Pending' indicator.

Case ID	Patient	Received	Type	Site	Status	Shared	Report
S11-00111	Doe, Jane	01/13/2011	Breast	Sunnyvale	In-Progress		
S11-00123	Smith, Mary	01/13/2011	Breast	Sunnyvale	In-Progress		
S11-00124	West, Tom	01/14/2011	Prostate	Sunnyvale	Released		
S11-00125	Jones, Laura	01/14/2011	Breast	Sunnyvale	Released		
S11-00127	Anderson, Bill	01/14/2011	Breast	Sunnyvale	Released		
S11-00134	Fish, Jenny	01/14/2011	Breast	Sunnyvale	Released		
S11-00138	Tyson, Diane	01/14/2011	Breast	Sunnyvale	Released		
S11-00139	Stevens, Rick	01/14/2011	Breast	Sunnyvale	Released		
S11-00142	Wayne, Berlo	01/14/2011	Breast	Sunnyvale	Released		
S11-00145	Clarkson, Patty	01/14/2011	Breast	Sunnyvale	Released		
S11-00146	Bregman, Joe	01/14/2011	Breast	Sunnyvale	Released		
S11-00148	Burnes, Sarah	01/14/2011	Breast	Sunnyvale	Released		
S11-00152	Jameson, Deb	01/14/2011	Breast	Sunnyvale	Released		
S11-00153	Cruz, Tom	01/14/2011	Breast	Sunnyvale	Released		
S11-00157	Blanch, Kate	01/14/2011	Breast	Sunnyvale	Released		
S11-00159	Wicks, Wanda	01/14/2011	Breast	Sunnyvale	Released		
S11-00161	Foye, Tim	01/14/2011	Breast	Sunnyvale	Released		
S11-00162	Hatch, Teresa	01/14/2011	Breast	Sunnyvale	Released		
S11-00164	Corne, Clara	01/14/2011	Breast	Sunnyvale	Released		
S11-00166	Haye, John	01/14/2011	Breast	Sunnyvale	Released		
S11-00168	Tully, Betty	01/14/2011	Breast	Sunnyvale	Released		

**Accession #: S11-00123**

Patent Name: Smith, Mary  
Age/Gender: 53 Year Old Female  
Primary Tissue Type: Breast

**Status of Slide (1) 3-ER**

**Station:** BenchMark

**Event:** Stain Protocol Started

**Event Date/Time:** 05/13/2011 10:20:54

**Run Estimated End Time:** 10:45

**Run Number:** 1141

**Slide Position:** 13

[Tracking History](#)

**Biopsy History**  
No prior history on record

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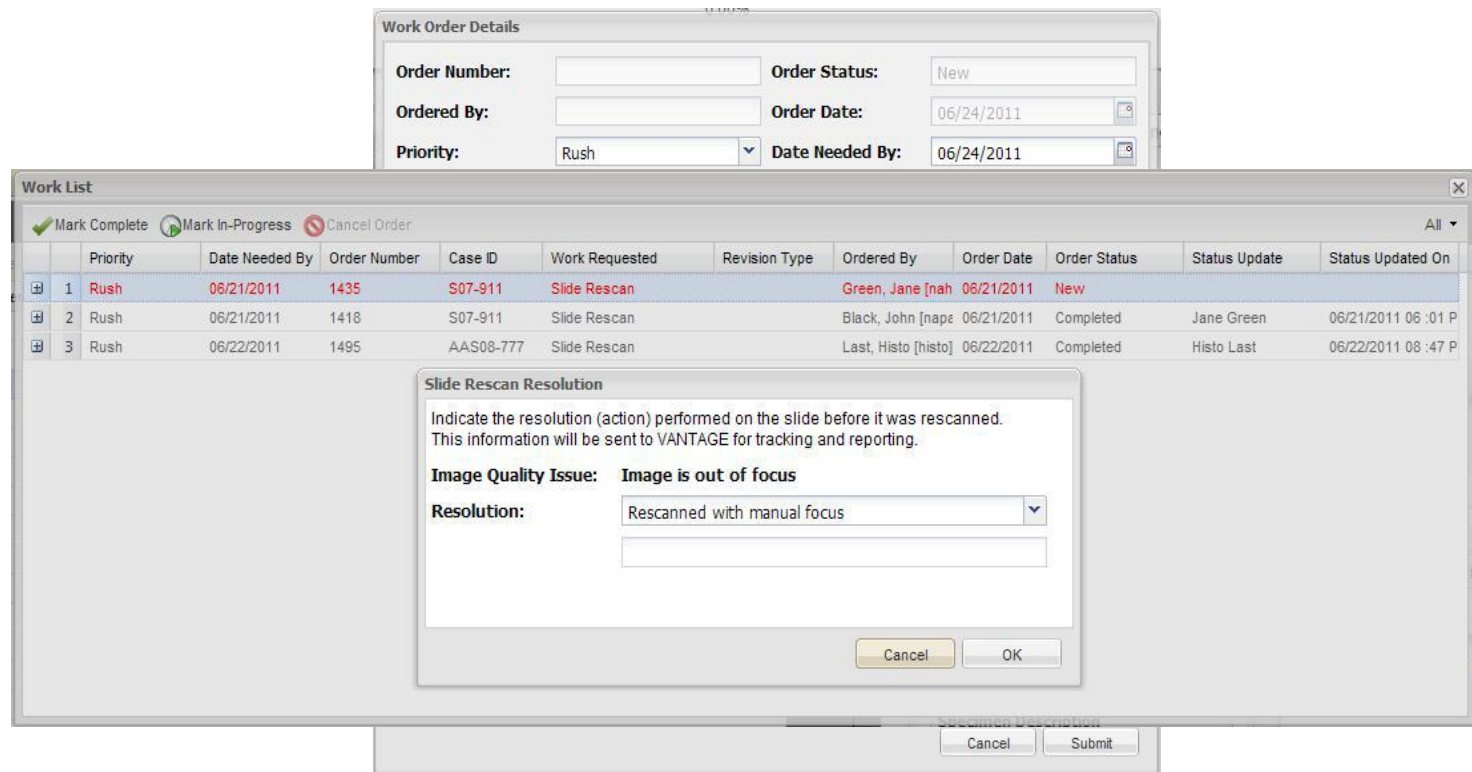
The screenshot displays a software interface with three main components:


- Work Order Details:** A form with the following fields:
  - Order Number:
  - Order Status:
  - Ordered By:
  - Order Date:
  - Priority:
  - Date Needed By:
- Work List:** A table with columns: Priority, Date Needed By, Order Number, Case ID, Work Requested, Revision Type, Ordered By, Order Date, Order Status, Status Update, Status Updated On.
 

Priority	Date Needed By	Order Number	Case ID	Work Requested	Revision Type	Ordered By	Order Date	Order Status	Status Update	Status Updated On
1	06/21/2011	1435	S07-911	Slide Rescan		Green, Jane [nah]	06/21/2011	New		
2	06/21/2011	1418	S07-911	Slide Rescan		Black, John [nape]	06/21/2011	Completed	Jane Green	06/21/2011 06 :01 P
3	06/22/2011	1495	AAS08-777	Slide Rescan		Last, Histo [histo]	06/22/2011	Completed	Histo Last	06/22/2011 08 :47 P
- Slide Rescan Resolution:** A dialog box with the following text and fields:
  - Indicate the resolution (action) performed on the slide before it was rescanned. This information will be sent to VANTAGE for tracking and reporting.
  - Image Quality Issue: **Image is out of focus**
  - Resolution:
  - Buttons: Cancel, OK

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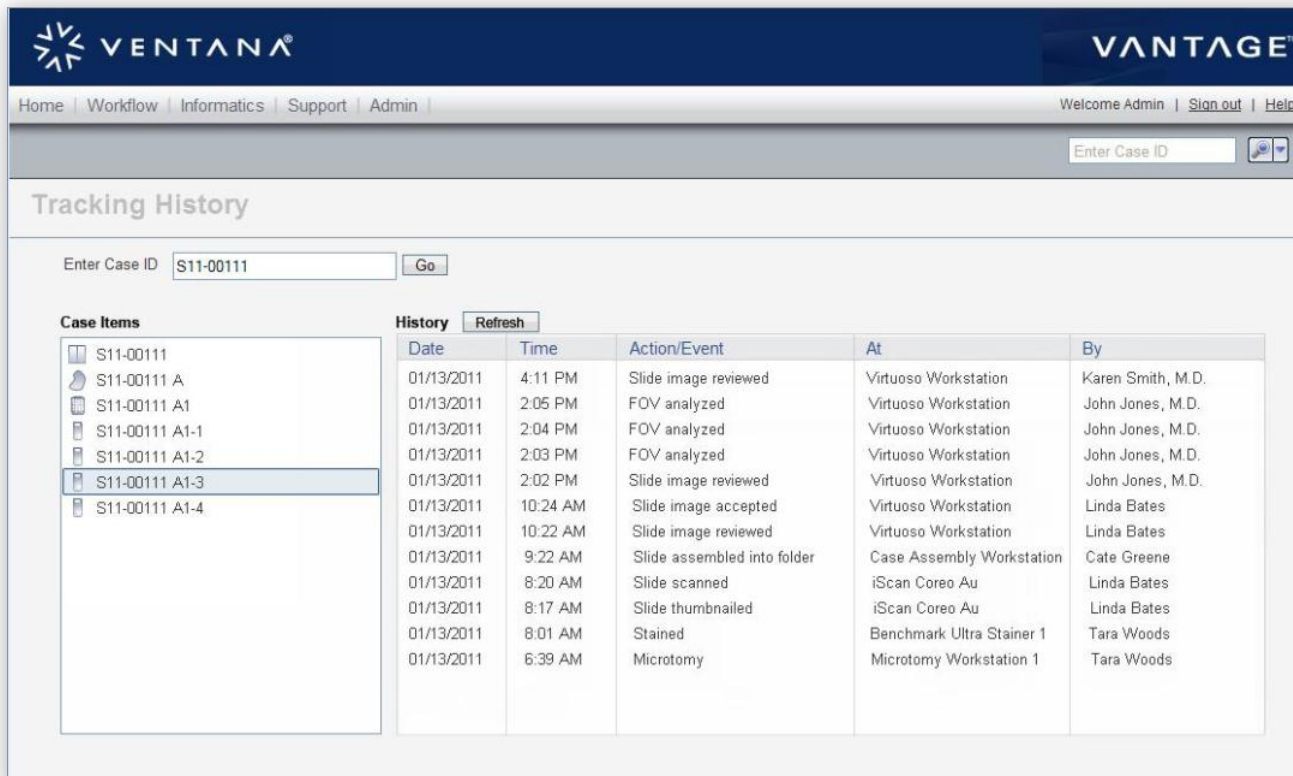The screenshot displays the VANTAGE web interface. At the top, the VENTANA logo is on the left and the VANTAGE logo is on the right. Below the logos is a navigation bar with links for Home, Workflow, Informatics, Support, and Admin. On the right side of the navigation bar, there is a user greeting 'Welcome Admin' and links for Sign out and Help. Below the navigation bar is a search field labeled 'Enter Case ID' with a search icon.

The main content area is titled 'Tracking History'. It features a search field with 'S11-00111' entered and a 'Go' button. Below the search field, there are two sections: 'Case Items' and 'History'.

The 'Case Items' section shows a list of items for case S11-00111:

- S11-00111
- S11-00111 A
- S11-00111 A1
- S11-00111 A1-1
- S11-00111 A1-2
- S11-00111 A1-3 (highlighted)
- S11-00111 A1-4

The 'History' section has a 'Refresh' button and a table with the following columns: Date, Time, Action/Event, At, and By. The table contains the following data:

Date	Time	Action/Event	At	By
01/13/2011	4:11 PM	Slide image reviewed	Virtuoso Workstation	Karen Smith, M.D.
01/13/2011	2:05 PM	FOV analyzed	Virtuoso Workstation	John Jones, M.D.
01/13/2011	2:04 PM	FOV analyzed	Virtuoso Workstation	John Jones, M.D.
01/13/2011	2:03 PM	FOV analyzed	Virtuoso Workstation	John Jones, M.D.
01/13/2011	2:02 PM	Slide image reviewed	Virtuoso Workstation	John Jones, M.D.
01/13/2011	10:24 AM	Slide image accepted	Virtuoso Workstation	Linda Bates
01/13/2011	10:22 AM	Slide image reviewed	Virtuoso Workstation	Linda Bates
01/13/2011	9:22 AM	Slide assembled into folder	Case Assembly Workstation	Cate Greene
01/13/2011	8:20 AM	Slide scanned	iScan Coreo Au	Linda Bates
01/13/2011	8:17 AM	Slide thumbnailled	iScan Coreo Au	Linda Bates
01/13/2011	8:01 AM	Stained	Benchmark Ultra Stainer 1	Tara Woods
01/13/2011	6:39 AM	Microtomy	Microtomy Workstation 1	Tara Woods

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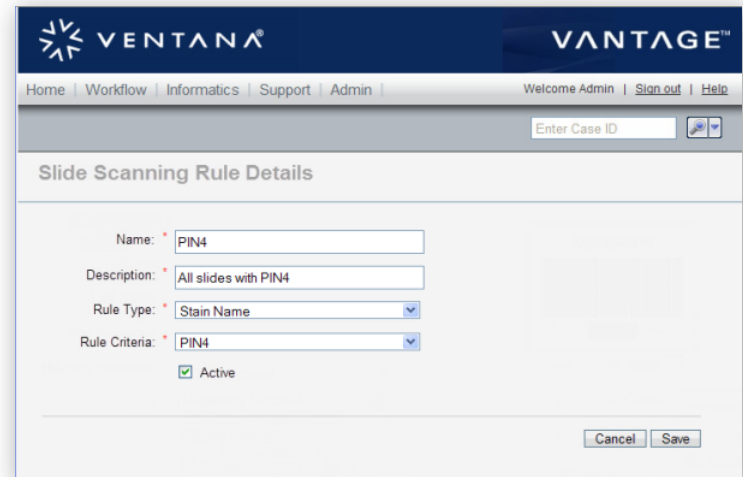
**Slide Scanning Rules**  
Create or manage rules on what slides to scan

[Add](#)

Scanning Rule	Description	Active		
Breast	Scan all tissue types: Breast cases	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
Dr. Jones	Scan all cases for: Dr. John Jones	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
PIN4	Scan all stain types: PIN4 cases	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>

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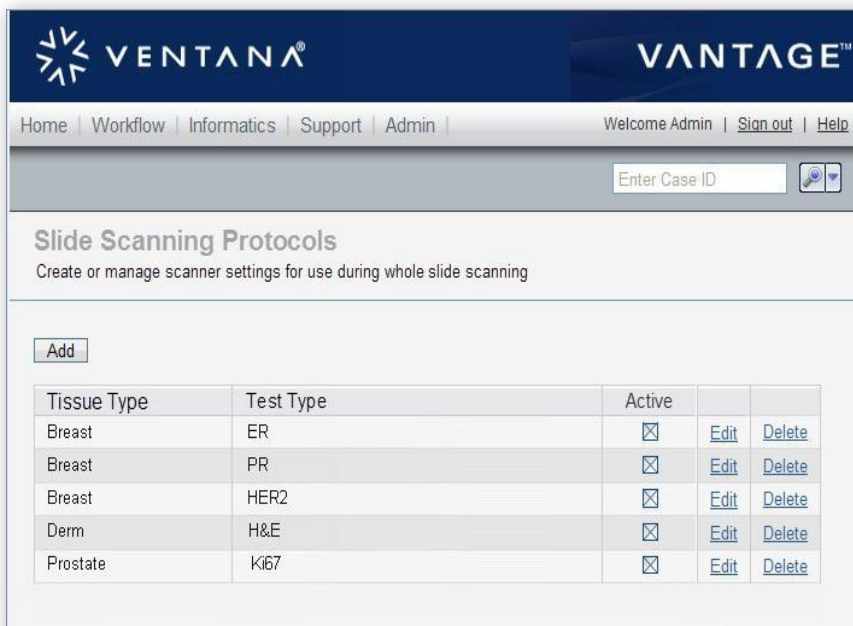
VENTANA VANTAGE™

Home | Workflow | Informatics | Support | Admin | Welcome Admin | Sign out | Help


Enter Case ID

### Slide Scanning Protocols

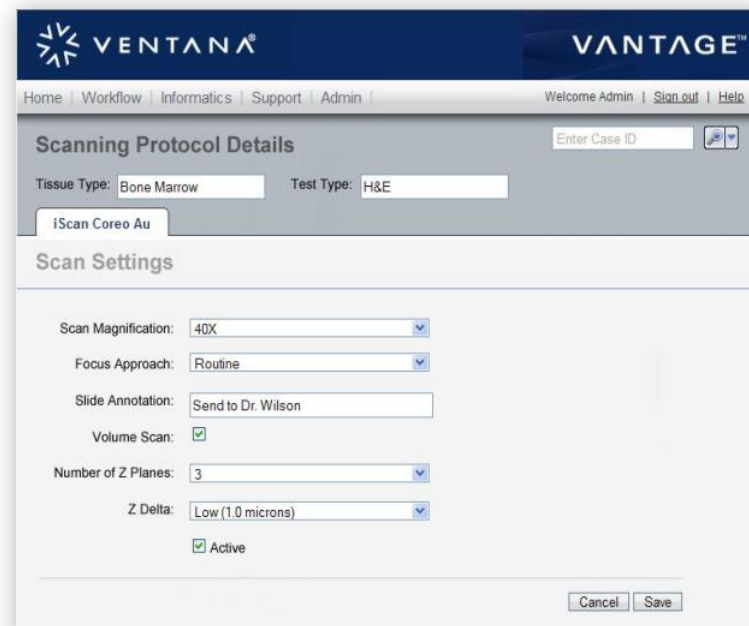Create or manage scanner settings for use during whole slide scanning

Add

Tissue Type	Test Type	Active		
Breast	ER	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
Breast	PR	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
Breast	HER2	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
Derm	H&E	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>
Prostate	Ki67	<input checked="" type="checkbox"/>	<a href="#">Edit</a>	<a href="#">Delete</a>

VENTANA VANTAGE™

Home | Workflow | Informatics | Support | Admin | Welcome Admin | Sign out | Help

Enter Case ID

### Scanning Protocol Details

Tissue Type: Bone Marrow Test Type: H&E

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